

Brun Medical Co., Ltd.

GRA-406002,Rev.02

**Manifold [INSTRUCTION FOR USE]**

1. INTENDED PURPOSE

The product is intended to establish multiple channels and provide liquid medicine or contrast medium injection as a connecting channel by professional physician.

2. INTENDED USER

Intended users are the competent physicians who have the training in interventional treatment.

3. INTENDED PATIENT POPULATION

Patients needing interventional treatment.

4. CONTRAINDICATION

No absolute contraindications when used correctly by medical personnel.

5. HANDING AND STORAGE

Stored in the relative humidity of not more than 80%, non-corrosive gases, a common, dry, well-ventilated and clean environment.

6. STRUCTURE AND SPECIFICATIONS

The device is consist of the luer connector, valve body, handle.

Specifications:

BR-MF1-OFF	BR-MF1-ON
BR-MF2-OFF	BR-MF2-ON
BR-MF3-OFF	BR-MF3-ON

7. PRODUCT PERFORMANCE

Adopt the transparent materials, manifolds' flow path are clearly visible in use

8. INSTRUCTION FOR USE

- 1.Remove the packaging, do not use if the package damaged.
- 2.Take out manifolds, and take off protective cap, then connect lines according to the regulation, check all the connections to make sure not liquid and air leakage.
- 3.According to patient needs, infuse appropriate medical liquid to implement diagnosis and treatment.
- 4.Single use only, discards after use.

9. PRODUCT LABEL AND PACKAGING (See the label or packaging)

- 1.Product name, model and specification.
- 2.Production date, batch lot.
- 3.Sterilization expiry date.

10. CAUTION

1. Procedure must be performed by trained medical personnel.
2. Sterilization is valid for three years, please confirm date of sterilization before use,do not use if expiry.
3. The device is designed and intended for SINGLE USE ONLY.
4. Do not resterilize and/or reuse, otherwise patients could get the risk of infection.
- 5.Sterilized by ethylene oxide. Do not use if the package damaged.




Caution



Use-by date



Keep dry

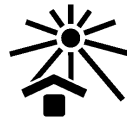

Keep away from sunlight



Catalogue number



Date of manufacture



Manufacturer



Do not re-use



Sterilized Using ethylene oxide



Do not use if package is damaged and consult instruction for use



Batch code



Unique device identifier



Authorized representative in the European Community/ European Union



Consult instructions for use or consult electronic instructions for use




Do not re-sterilize



Medical device



Single sterile barrier system

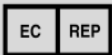

Cnovate Medical B.V.

ADDRESS: Terminalweg 15, 3821AJ Amersfoort, The Netherlands

TEL: +31 850140404

E-mail: [cs@cnovate.eu](mailto:cs@cnovate.eu)



Brun Medical Co., Ltd.

ADDRESS: Building B8, Entrepreneurship and Innovation Park.88 Changtang Road, Yunlong Demonstration Zone,Zhuzhou, 412000 Hunan ,P.R.China

TEL: +0731-22029666

WEB: [www.brunmd.com](http://www.brunmd.com)

Date of issue:2024/05/13